



TETRAGRAPH

Quick Guide

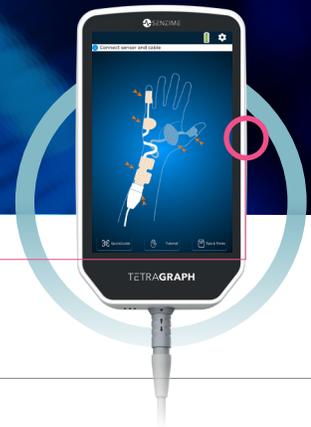

- 1 Turn on Monitor**
If the monitor is not connected to power, ensure adequate battery charge.

- 2 Place Sensor**
Prepare skin, place stimulating electrodes on ulnar nerve, attach recording electrode to Abductor Digiti Minimi (ADM) muscle or Adductor Pollicis (AP) muscle, and position grounding electrode away from recording electrode.

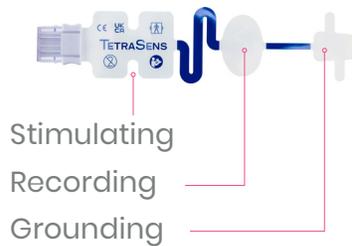

ADM



AP



Foot Placement

- 3 Connect Cable to Sensor**
A quick, single click sound confirms a proper connection.



- 4 Press Auto Play Button**
Press auto play button before administering muscle relaxants for optimal sensor placement feedback.

The monitor will auto-select optimal stimulus and set a signal strength. Strive for signal strength >5 mV indicated in green.



- 5 Enter TetraGraph® Adaptive PTC™**
TetraGraph® Adaptive PTC™ Enters Deep Block Mode automatically when TOFC is 0.



- 6 Ensure Acceptable Recovery**
The patient has reached acceptable recovery when TOFR is $\geq 90\%$.



Important:

Please read the complete TetraGraph User Manual, including warnings and cautions section, before operating the device to ensure safe and correct use.